## Parcus 3.5mm Hex Driver with Handle

**Important Product Information**  
Produktinformationsblatt  
Información Importante sobre el Producto  
Informations Importantes sur le Produit  
Informazioni importanti sul prodotto

**Directions for Use**  
Gebrauchsanleitung  
Instrucciones de uso  
Mode d'emploi  
Istruzioni per l'uso

---

30385 Rev 7  
Eff. 12/22

**ANIKA**

Mfd by:  
Parcus Medical, LLC  
6423 Parkland Dr.  
Sarasota, FL 34243

[www.anika.com](http://www.anika.com)  
(941)755-7965

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## Parcus 3.5mm Hex Driver with Handle

### 1. Indications:

The Parcus 3.5mm Hex Driver with Handle is designed to insert and remove Parcus Titanium Interference Screws. The driver features a 1.5mm cannulation and is compatible with Parcus 1.5mm Guide Wires.

### 2. Warnings:

- A. This product is intended for use by or on the order of a physician.
- B. This product should not be used to implant any device other than Parcus Titanium Interference Screws.
- C. Do not use this product if it is bent or there is visible damage to the shaft or drive section.

### 3. Material:

This device is comprised of a Stainless Steel Alloy shaft and an Aluminum handle. The materials used in the manufacture of this device that are intended to be placed inside the body are radio-opaque and can, therefore, be detected with conventional X-Ray or fluoroscopy.

### 4. Inspection:

- A. Inspect the device for damage at all stages of handling.
- B. If damage is detected, consult the manufacturer for guidance.

### 5. Cleaning:

- A. Immediate rinsing and cleaning after use with an enzymatic detergent will effectively remove and prevent drying of adherent blood, tissue, etc.
- B. Scrub device with a soft brush, paying special attention to the cannulation and other areas where debris might accumulate. Always avoid any harsh materials that can scratch or mar the surface of the device.
- C. Rinse the device thoroughly with water following the cleaning process.

### 6. Sterilization:

This device is not provided sterile. The following table provides recommended minimum sterilization parameters that have been validated by Parcus Medical to provide a 10<sup>-6</sup> sterility assurance level (SAL):

Cycle Type	Minimum Temperature	Minimum Exposure Time (Wrapped)	Minimum Dry Time
Gravity	132°C / 270°F	15 minutes	30 minutes <sup>1</sup>
Pre-vacuum	132°C / 270°F	4 minutes	

<sup>1</sup> Drying times vary according to load size and should be increased for larger loads.

### 7. Directions for use:

- A. Place a Parcus Titanium Interference Screw onto the drive portion of the driver until it is fully engaged.
- B. Place the screw and driver over an appropriately placed Parcus 1.5mm Guide Wire to guide the screw into the insertion site.
- C. Rotate the driver clockwise until the screw has reached the desired depth. A counterclockwise rotation of the driver will remove the screw.
- D. Remove the driver from the screw, followed by removal of the guide wire.

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