## Parcus PEEK CF Interference Screw Driver with Handle

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---

**Parcus PEEK CF Interference Screw Driver with Handle  
Product Information Sheet and Directions for Use:**

**1. Indications:**

The Parcus PEEK CF Interference Screw Driver with Handle is designed to insert and remove Parcus PEEK CF Interference Screws. The driver features a 1.5mm cannulation and is compatible with Parcus 1.5mm Guide Wires.

**2. Warnings:**

- A. This product is intended for use by or on the order of a physician.
- B. This product should not be used to implant any device other than Parcus PEEK CF Interference Screws.
- C. Do not use this product if it is bent or there is visible damage to the shaft or drive section.

**3. Material:**

This device is manufactured from a Stainless Steel Alloy shaft and an Aluminum handle. The materials used in the manufacture of this device that are intended to be placed inside the body are radio-opaque and can, therefore, be detected with conventional X-Ray or fluoroscopy.

**4. Inspection:**

- A. Inspect the device for damage at all stages of handling.
- B. If damage is detected, consult the manufacturer for guidance.

**5. Cleaning:**

- A. Immediate rinsing and cleaning after use with an enzymatic detergent will effectively remove and prevent drying of adherent blood, tissue, etc.
- B. Scrub device with a soft brush, paying special attention to the cannulation and other areas where debris might accumulate. Always avoid any harsh materials that can scratch or mar the surface of the device.
- C. Rinse the device thoroughly with water following the cleaning process.

**6. Sterilization:**

This product is a non-sterile item that must be cleaned and sterilized prior to use. The following table provides recommended minimum sterilization parameters that have been validated by Parcus Medical to provide a  $10^{-6}$  sterility assurance level (SAL):

Cycle Type	Minimum Temperature	Minimum Exposure Time (Wrapped)	Minimum Dry Time
Gravity	132°C / 270°F	15 minutes	30 minutes <sup>1</sup>
Pre-vacuum	132°C / 270°F	4 minutes	

<sup>1</sup> Drying times vary according to load size and should be increased for larger loads

**7. Directions for use:**

- A. Place a Parcus PEEK CF Interference Screw onto the drive portion of the driver until it is fully engaged.
- B. Place the screw and driver over an appropriately placed Parcus 1.5mm Guide Wire to guide the screw into the insertion site.
- C. Rotate the driver clockwise until the screw has reached the desired depth. A counterclockwise rotation of the driver will remove the screw.
- D. Remove the driver from the screw, followed by removal of the guide wire.

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