**Parcus Cannulated Drills  
and Headed Reamers**

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---

**Parcus Cannulated Drills and Headed Reamers**  
**Product Information Sheet and Instructions for Use (English)**

**1. Indications:**

Parcus Cannulated Drills and Headed Reamers are used to create tunnels and sockets in bone. This product is designed to be used in conjunction with a 2.4mm guide pin.

**2. Warnings:**

- A. Caution: Federal Law restricts this device to sale by or on the order of a physician.
- B. Never place this product over a bent or damaged guide pin.
- C. Placing off-axis loads on the guide pin by deflecting the drill/reamer may cause damage to both the drill/reamer and the guide pin.
- D. Never use a dull or damaged drill/reamer.
- E. This product is not intended to be left in the body.
- F. Be sure to remove any debris from the cannulation of the drill/reamer during the cleaning process.

**3. Material:**

These products are manufactured from a stainless steel alloy. The material used in the manufacture of the devices that are intended to be placed inside the body are radio-opaque and can, therefore, be detected with conventional X-Ray or fluoroscopy.

**4. Inspection:**

- A. Inspect the device for damage at all stages of handling.
- B. If damage is detected, consult the manufacturer for guidance.

**5. Cleaning:**

- A. Immediate rinsing and cleaning after use with an enzymatic detergent will effectively remove and prevent drying of adherent blood, tissue, etc.
- B. Scrub device with a soft brush, paying special attention to the cannulation and other areas where debris might accumulate. Always avoid any harsh materials that can scratch or mar the surface of the device.
- C. Rinse the device thoroughly with water following the cleaning process.

**6. Sterilization:**

This product is a non-sterile item that must be cleaned and sterilized prior to use. The following table provides recommended minimum sterilization parameters that have been validated by Parcus Medical to provide a  $10^{-6}$  sterility assurance level (SAL):

Cycle Type	Minimum Temperature	Minimum Exposure Time (Wrapped)	Minimum Dry Time
Gravity	132°C / 270°F	15 minutes	30 minutes <sup>1</sup>
Pre-vacuum	132°C / 270°F	4 minutes	

<sup>1</sup> Drying times vary according to load size and should be increased for larger loads

**7. Instructions for use:**

- A. Use a purpose designed drill motor or wire driver to place a guide pin in the desired position.
- B. Securely attach the drill/reamer into a drill chuck.
- C. Place the drill/reamer over the guide pin.
- D. Make sure that the drill motor is set to rotate the drill/reamer in a clockwise direction.
- E. Prudently advance the drill/reamer over the guide pin, ensuring to maintain a coaxial orientation between the pin and the drill/reamer.
- F. After the desired depth has been reached, reverse the drill motor and remove the drill/reamer from the bone.

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