**Parcus Drill Bits  
for Suture Anchor Placement**

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---

## **Parcus Drill Bits for Suture Anchor Placement**

### **1. Indications:**

Parcus Drill Bits are provided in various lengths and diameters as complements to suture anchors and/or Drill Guides. They are used to create sockets and tunnels in bone.

### **2. Warnings:**

- A. This product is intended for use by or on the order of a physician.
- B. This product is designed for single use only and should not be re-sterilized. Drill Bits that are reused can fatigue and break.
- C. Never use a bent Drill Bit.
- D. This product is not intended for use as an implant.
- E. Loading the Drill Bit in an off axis manner may result in breakage or bending.

### **3. Material:**

This device is manufactured from a Stainless Steel Alloy. The materials used in the manufacture of this device that are intended to be placed inside the body are radio-opaque and can, therefore, be detected with conventional X-Ray or fluoroscopy.

### **4. Sterilization:**

This device is provided in sterile packaging. The contents are sterilized by EO gas. This product must never be re-sterilized.

### **5. Packaging and Labeling:**

- A. Do not use this product if the packaging or labeling has been damaged, shows signs of exposure to moisture or extreme temperature or has been altered in any way.
- B. Please contact Parcus Medical Customer Service to report any package damage or alterations.

### **6. Storage:**

Products must be stored in the original unopened package in a dry place and must not be used beyond the expiration date indicated on the package.

### **7. Directions for use:**

- A. Place the Drill Bit into the chuck of a purpose designed drill or wire driver. Drill Bits may have laser etched lines to help determine depth of penetration. Attach Drill Bits so that laser lines are visible as required.
- B. If a Drill Guide is utilized, advance the Drill Bit through the guide and into the bone to the required depth.
- C. Remove the Drill Bit from the guide.
- D. Remove the Drill Guide, as necessary.

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