Anika Drill Guide for Suture Anchor Placement
Guía de broca para colocación de anclajes de sutura Anika
Guide de perçage Anika pour placement d'ancre de suture
Anika Bohrführung zum Platzieren eines Fadenankers
Guida per trapano Anika per il posizionamento dell'ancora di sutura
Guia de Perfuração Anika para Posicionamento de Âncoras de Sutura
Οδηγός Τρυπανιού Anika για Τοποθέτηση Αγκιστρων Ραμμάτων
„Anika“ grąžto kreiptuvas siūlo inkarų įvedimui
Anika drillfører for plassering av suturanker
Anika borrhguide för placering av suturankare
Ghidaj burghiu Anika pentru plasarea ancorei de sutură
Prowadnica wiertła do umieszczania kotwicy do szwów Anika
Водач за свредла за поставяне на шевни анкери Anika

Important Product Information
Información importante sobre el producto
Informations importantes relatives au produit
Wichtige Produktinformationen
Informazioni importanti sul prodotto
Informações Importantes sobre o Produto
Σημαντικές πληροφορίες για το προϊόν
Svarbi informacija apie gaminį
Viktig produktinformasjon
Viktig produktinformation
Informații referitoare la produs
Ważne informacje o produkcie
Важна информация за продукта

Directions for Use
Instrucciones de uso
Instructions d'utilisation
Gebrauchsanweisung
Istruzioni per l'uso
Instruções para o uso
Οδηγίες χρήσης
Naudojimo instrukcija
Bruksanvisning
Anvisningar för användning
Instrucțiuni de utilizare
Wskazówki dotyczące użytkowania
Инструкции за употреба

30532 Rev 10
Eff. 07/24

Parcus Medical, LLC
6423 Parkland Dr.
Sarasota, FL 34243 USA

 **ANIKA** 
www.anika.com
(941)755-7965
MDSS GmbH
Schiffgraben 41
30175 Hanover, Germany

2797
EC REP

Anika Drill Guide for Suture Anchor Placement (EN)

1. Intended Use:

Anika Drill Guides are provided in various configurations to assist in the placement of suture anchors and/or drill bits.

2. Warnings:

- A. Caution: Federal law restricts this device to sale by or on the order of a physician.
- B. Anika Drill Guides are supplied unsterile as reusable devices.
- C. Never use a bent or damaged drill guide or obturator.
- D. This product is not intended for use as an implant.
- E. Loading this instrument or any instrument used with this device in an off-axis manner may result in breakage or bending of the instrument, drill guide or obturator.

3. Material:

The materials used in the manufacture of this device that are intended to be used inside the body are Stainless Steel Alloy and are, therefore, radio-opaque and can be detected with conventional X-Ray or fluoroscopy. Other parts of the device that are not used inside the body may be manufactured from Stainless Steel Alloy, Aluminum or Plastic. No part of this device is intended to be implanted in the body.

4. Inspection:

- A. Inspect the device for damage at all stages of handling.
- B. If damage is detected, consult the manufacturer for guidance.

5. Cleaning:

- A. Completely submerge used instruments in enzyme solution and allow to soak for 20 minutes. Use a soft bristled, nylon brush to gently scrub the instrument until all visible soil has been removed. Particular attention must be given to crevices, lumen, mated surfaces, connections and other hard-to-clean areas. Lumens should be cleaned with a long, narrow, soft bristled brush (i.e. pipe cleaner brush).
- B. Remove instrument from enzymatic solution and rinse in tap water for a minimum of 3 minutes. Thoroughly and aggressively flush lumens, holes, and other difficult-to-reach areas.
- C. Place prepared cleaning agents in a sonication unit. Completely submerge instrument in cleaning solution and sonicate for 10 minutes at 45-50 kHz.
- D. Rinse instrument in purified water for at least 3 minutes or until there is no sign of blood or soil on the instrument or in the rinse stream. Thoroughly and aggressively flush lumens, holes, and other difficult-to-reach areas.
- E. Repeat steps A through D a second time and ensure the instrument is free of blood or soil before moving to step F.
- F. Remove excess moisture from the instrument with a clean, absorbent, and non-shedding wipe.

6. Sterilization:

This device is not provided sterile. Devices shall be double wrapped in wrapping material or pouches, double wrapped while in a container or tray, or placed within a reusable sterilization container. Combining two or more systems may hinder the efficacy of the sterilization to an unacceptable level. Reusable devices shall be sterilized with a load size which does not exceed 10,630 grams. The following table provides recommended minimum sterilization parameters that have been validated by Anika Therapeutics to provide a 10⁻⁶ sterility assurance level (SAL):

Cycle Type	Minimum Temperature	Minimum Exposure Time (Wrapped)	Minimum Dry Time
Pre-vacuum	132°C / 270°F	4 minutes	30 minutes ¹
	134°C / 273°F	3 minutes	
	132°C / 270°F	18 minutes ²	50 minutes ¹

¹ Drying times vary according to load size and should be increased for larger loads.

² This cycle is not for use in the United States. These are disinfection/steam sterilization parameters recommended by the World Health Organization (WHO) for reprocessing instruments where there is a concern regarding TSE/CDJ contamination. This cycle is not to be used for the inactivation of TSE/CJD contamination.

7. Instructions for use:

- A. Place the Drill Guide with obturator, as applicable, on the targeted bone through an open incision, percutaneously, or through an arthroscopic cannula.
- B. Remove obturator, as applicable, and advance the required instrumentation through the guide to the desired depth.

- C. Remove the instrumentation from the guide.
- D. Remove the Drill Guide, as necessary.

8. Adverse/ Reportable Event:

Any serious incident that occurs in relation to the device should be reported to both Anika Therapeutics and the applicable authority using the following contact information.

Country	Authority	Email	Website
All	Anika Therapeutics	ParcusCS@Anika.com	www.anika.com
Australia	Therapeutic Goods Administration (TGA)	IRIS@health.gov.au	www.tga.gov.au/reporting-adverse-events
European Union	EC Representative	info@mdssar.com	www.mdssar.com

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