Anika Tibial Plug
Tapón tibial Anika
Fiche tibiale Anika
Anika Tibiapfropfen
Collegamento tibiale Anika
Tampão Tibial Anika
Κνημιαίο Βύσμα Anika
„Anika“ blauzdkaulio kištukas
Anika skinnebeinplugg
Anika lårbensplugg
Dop tibial Anika
Korek piszczelowy Anika
Тибіална тапа Anika

Important Product Information
Información importante sobre el producto
Informations importantes relatives au produit
Wichtige Produktinformationen
Informazioni importanti sul prodotto
Informações Importantes sobre o Produto
Σημαντικές πληροφορίες για το προϊόν
Svarbi informacija apie gaminį
Viktig produktinformasjon
Viktig produktinformation
Informații referitoare la produs
Ważne informacje o produkcie
Важна информация за продукта

Directions for Use
Instrucciones de uso
Instructions d'utilisation
Gebrauchsanweisung
Istruzioni per l'uso
Instruções para o uso
Οδηγίες χρήσης
Naudojimo instrukcija
Bruksanvisning
Anvisningar för användning
Instrucțiuni de utilizare
Wskazówki dotyczące użytkowania
Инструкции за употреба

30966 Rev 7
Eff. 08/24

 Parcus Medical, LLC
6423 Parkland Dr.
Sarasota, FL 34243 USA

 **ANIKA**
www.anika.com
(941)755-7965

 2797
EC REP
MDSS GmbH
Schiffgraben 41
30175 Hanover, Germany

Anika Tibial Plug (EN)

1. Intended Use:

Anika Tibial Plugs are used during ACL/PCL ligament repair or reconstruction procedures to limit the loss of irrigation fluid from the knee joint. The Anika Tibial Plugs are supplied in two configurations; solid and with a 2.4mm cannulation. Both designs are tapered to accommodate tunnels from 7mm to 12mm in diameter.

2. Warnings:

- A. This product is intended for use by or on the order of a physician.
- B. Do not use this product if it shows any visible signs of damage.
- C. This device is not intended to be used to dilate the tibial tunnel. Care should be exercised when placing the plug to prevent fracture of the proximal tibia.
- D. This device is not intended to be used as an implant.
- E. Placing off-axis loads on the guide pin by deflecting the tibial plug may cause damage to both the tibial plug and the guide pin.
- F. Caution: Common risks consistent with surgery may include: bone/tissue damage, revision surgery, soft tissue irritation, infection, allergic reactions.

3. Material:

This device is comprised of PEEK, is radiolucent, and may be visualized through the use of ultrasound or MRI.

4. Inspection:

Instruments intended to be reused should be inspected prior to each use to assess damage and wear. Signs of damage and wear include surface discoloration, surface corrosion, surface damage, and general bending, cracking, and fractures. Cutting, drilling, and other instruments intended to have sharp edges will become dull over time, diminishing their ability to function as designed. Review sharp instruments for dullness or damage at the cutting/sharp feature. If the instrument does not pass inspection, do not use the instrument, and notify an Anika representative.

5. Storage:

Products must be stored in a dry place.

6. Cleaning:

- A. Completely submerge used instruments in enzyme solution and allow to soak for 20 minutes. Use a soft bristled, nylon brush to gently scrub the instrument until all visible soil has been removed. Particular attention must be given to crevices, lumen, mated surfaces, connections and other hard-to-clean areas. Lumens should be cleaned with a long, narrow, soft bristled brush (i.e. pipe cleaner brush).
- B. Remove instrument from enzymatic solution and rinse in tap water for a minimum of 3 minutes. Thoroughly and aggressively flush lumens, holes, and other difficult-to-reach areas.
- C. Place prepared cleaning agents in a sonication unit. Completely submerge instrument in cleaning solution and sonicate for 10 minutes at 45-50 kHz.
- D. Rinse instrument in purified water for at least 3 minutes or until there is no sign of blood or soil on the instrument or in the rinse stream. Thoroughly and aggressively flush lumens, holes, and other difficult-to-reach areas.
- E. Repeat steps A through D a second time and ensure the instrument is free of blood or soil before moving to step F.
- F. Remove excess moisture from the instrument with a clean, absorbent, and non-shedding wipe.

7. Sterilization:

This device is not provided sterile. Devices shall be double wrapped in wrapping material or pouches, double wrapped while in a container or tray, or placed within a reusable sterilization container. Combining two or more systems may hinder the efficacy of the sterilization to an unacceptable level. Reusable devices shall be sterilized with a load size which does not exceed 10,630 grams. The following table provides recommended minimum sterilization parameters that have been validated by Anika Therapeutics to provide a 10^{-6} sterility assurance level (SAL):

Cycle Type	Minimum Temperature	Minimum Exposure Time (Wrapped)	Minimum Dry Time
Pre-vacuum	132°C / 270°F	4 minutes	30 minutes ¹
	134°C / 273°F	3 minutes	

	132°C / 270°F	18 minutes ²	50 minutes ¹
--	---------------	-------------------------	-------------------------

¹ Drying times vary according to load size and should be increased for larger loads.

² This cycle is not for use in the United States. These are disinfection/steam sterilization parameters recommended by the World Health Organization (WHO) for reprocessing instruments where there is a concern regarding TSE/CDJ contamination. This cycle is not to be used for the inactivation of TSE/CJD contamination.

After sterilization aseptically transfer device into the sterile field in accordance with the institution's policy.

8. Directions for use:

- Aseptically transfer device into sterile field and create a tibial tunnel that is 7mm to 12mm in diameter.
- If a guide wire is present in the tunnel, select the cannulated version of the Anika Tibial Plug. The cannulation is 2.4mm in diameter. If no guide wire is present, the non-cannulated Plug can be used.
- Gently press the tapered end of the Plug into the tunnel until an interference fit, sufficient to keep the Plug in place, is achieved.
- Remove the Plug to pass instruments, graft material and suture.
- Replace the Plug, described in Step C, as necessary.

9. Device Disposal:

Devices that have evidence of damage must be properly disposed of in accordance with the institution's policy.

10. Adverse/ Reportable Event:

Any serious incident that occurs in relation to the device should be reported to both Anika Therapeutics and the applicable authority using the following contact information.

Country	Authority	Email	Website
All	Anika Therapeutics	ParcusCS@Anika.com	www.anika.com
Australia	Therapeutic Goods Administration (TGA)	IRIS@health.gov.au	www.tga.gov.au/reporting-adverse-events
European Union	EC Representative	info@mdssar.com	www.mdssar.com

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